



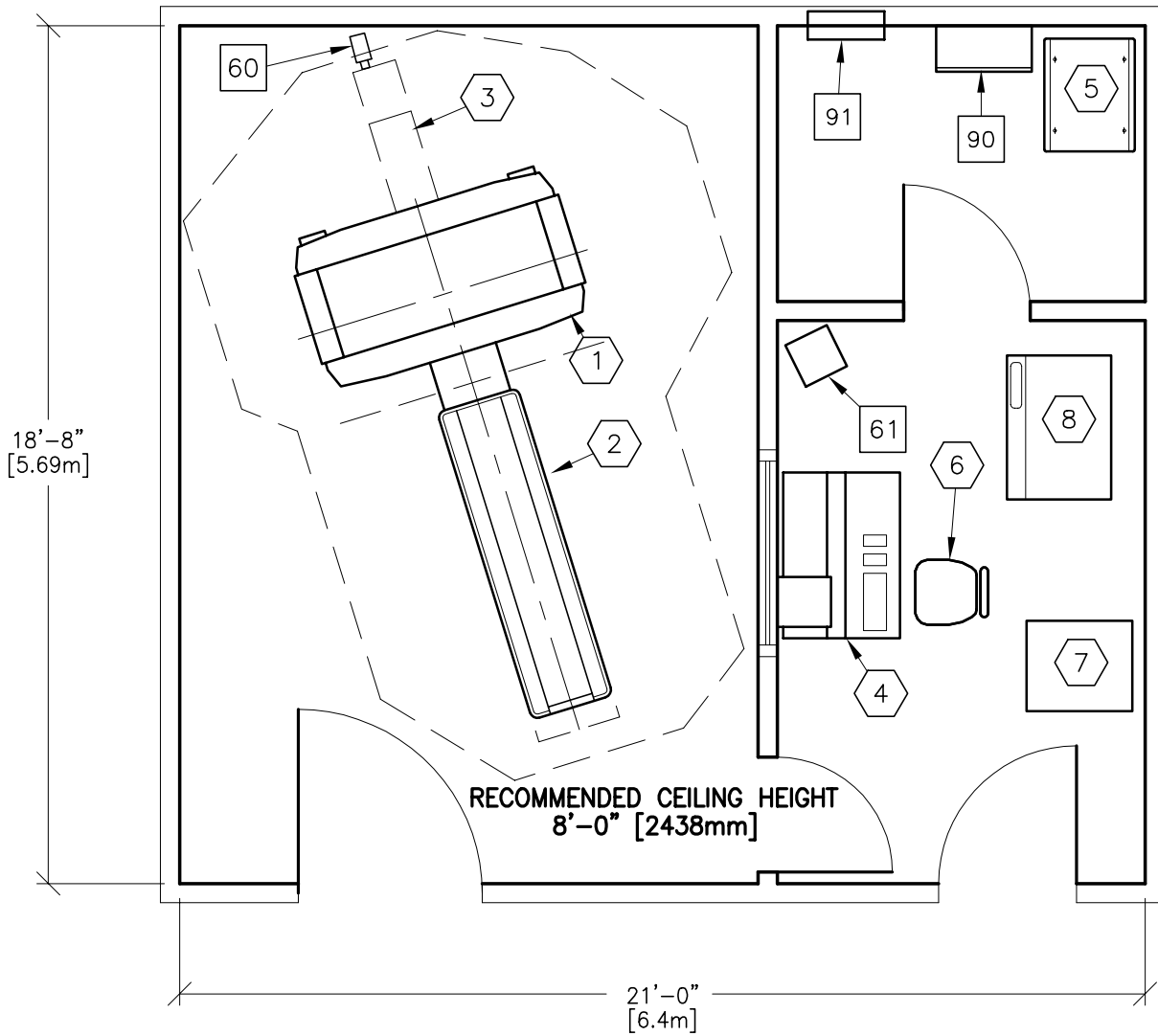

CT SYTEC 2/3/4000

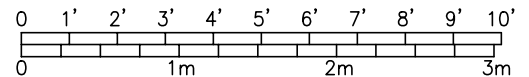

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
MINIMUM:	19'-6" x 17'-0" [5.94m x 5.18m]	7'-6" [2.28m]



SEE APPENDIX "A" FOR POWER SPECIFICATIONS

ROOM PLAN LOCATED ON PAGE 6-8

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	CT SYTEC 2/3/4000 GANTRY	1716 lb 779 kg	5955 btu 1745 W
②	1	PATIENT TABLE WITH EXTENDED TABLE TOP.	800 lb 363 kg	
③	1	TABLE EXTENDED THROUGH GANTRY		
④	1	2/3/4000 OPERATOR'S CONSOLE	507 lb 230 kg	5956 btu 1745 W
⑤	1	X-RAY GENERATOR (THIS UNIT MUST BE LOCATED A MINIMUM OF 6'-0" FROM THE GANTRY AND/OR CONSOLE).	755 lb 343 kg	794 btu 233 W
⑥	1	OPERATOR'S CHAIR		
⑦	1	MAGNETIC TAPE DRIVE UNIT	242 lb 110 kg	1190 btu 350 W
⑧	1	969 LASER IMAGER SYSTEM	610 lb 277 kg	2520 btu 739 W
***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****				
60		CLOSED CIRCUIT TV CAMERA		
61		CLOSED CIRCUIT TV MONITOR		
90		PowerTECH SUPPRESSOR/FILTER, WALL MOUNTED - GEMSG CAT. NO. R4502NG	168 lb 76 kg	300 btu 88 W
91		MAIN DISCONNECT CONTROL, - WALL MOUNTED - GE CAT NO R4502AD, 90 LBS		

ENVIRONMENT

Ambient Operation Temperature: 60-75° F (16° TO 24° C)
 Allowable Temperature Change: 5° F (3° C) per hour
 Humidity: 30-60% non condensing
 Allowable Humidity Change: 10% per hour
 Max - Altitude: 10,000 ft. above sea level